



Easy Quick TEST DEA 1

Individual test package contains:

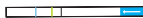

1 membrane

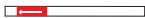

1 blood collector strip



1 test tube



1 buffer

1

Write down the name of the patient.




2



Add 3 of buffer in the test tube.


3

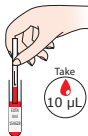

Take 10 µL with the blood collector strip.

Patient blood tube*


4

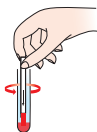

Add the blood collector strip in the test tube

and mix 7 sec.

5

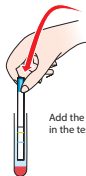

Add the membrane in the test tube.

6



Wait until complete migration




2 to 4 min.

7



To interpret the result, stick the membrane.

Read immediately after testing

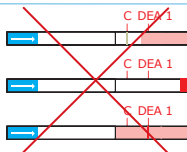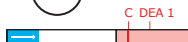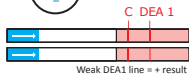

* Whole blood or packed red blood cells
ACD/CPD/EDTA
Do not use Heparin